



INSTRUCTIONS FOR USE

SURGICAL GOWN , CLASS I

MODEL: ALIVE PPE 9072

PURPOSE: Alive PPE one-piece surgical gown is a protective clothing with limited life, Class 1 protection

Surgical gowns manufactured and designed for use to prevent the transmission of infective agents between clinical staff and patients during surgical and other invasive procedures.

It is approved that the product fulfills all essential requirements and related rules of 9342/EEC Medical Devices Directive (MDD) Class 1 are applied.

PROPERTIES:

Cuff: Knitted Elastics
Neck Closure: Velcro neck closure for better adjustments to several people.
Belts: 2 ties system for outside.
Fabric: Endless Filaments Meltbown Thermally Bonded. %100 polypropylene 25 gr/m2,

LIMITATIONS:

Exposure to certain Chemicals or high concentrations may require higher protective properties through the material or construction properties of the surgical gown. The user will be the only one to assess compliance with the required type of protection and proper connection of the suit with additional protective equipment.

METHOD OF PUTTING ON AND USING THE NON-STERIL SURGICAL GOWN:

Before use, check that the selected product is suitable for the existing hazard and in the correct size.

Remove the gown from the packaging.

Check visually that the isolation gown is intact (no piercing, no torn seams, etc.)

Wear first on sleeves, close/adjust the velcro on back neck, tie the back belts.

WARNINGS:

The surgical gown is intended for single use and should be discarded after use. Accepted configuration cannot be modified or changed.

If tears, punctures etc. occur, immediately leave the workplace and put on a new gown. Do not remove the gown if the user is still in the danger area.

Surgical gown with electrostatic properties must not be opened or removed when in a potentially explosive or flammable atmosphere or carrying explosive, flammable substances.

To provide complete protection, all openings should be closed/sealed.

Extended wearing of surgical gown can cause heat stress.

Overheating and discomfort can be reduced or eliminated by using appropriate undergarments or appropriate ventilation equipment.

The product is flammable - keep away from fire.

TRANSPORT, STORAGE AND DISPOSAL:

The product must be transported and stored in the original packaging in dry rooms away from sources of light and heat. If the suits are not contaminated, they can be disposed of with solid municipal waste. If they have been contaminated, they must be disposed of in accordance with applicable laws and regulations.

USETIME:

It is suggested to use the product within a period of five years from the date of production written on label. Month and year of production: MM/YYYY

SIZES AND DIMENSIONS OF BODY (CM) -

	L	XL
Fr.Length from HPS	115,0	117,0
1 / 2 Bottom Hem	75,0	79,0
Sleeve Length	56,5	57,5
Cuff	6,0	6,0

PRODUCT LABEL (EXAMPLE):

Producer: **NORKONTEKS**
TEKSTİL İHR.İTH.SAN. VE TİC.A.Ş.

Dogus Cd, 3/19 Sk, No:12, Begos 35160,
Buca/İZMİR/TURKEY - 0090-232-4408686

Style Code: ALIVE PPE 9072
Surgical Gown Class I



MAINTENANCE AND CLEANING:



EU DECLARATION OF CONFORMITY:

European Regulation(EU) 2017/745 and 93/42/EEC Medical Devices Directive establishing technical requirements for medical devices , in effective wording



L



XL